

PW 6163, PW 6243, PW 6323


de	Montageanweisung BWS 02 Schaumkompensator (Bausatz Wrasen- und Schaumableitung) Reine-Unreine-Seite	en	Fitting instructions - BWS 02 foam compensator (vapour and foam dissipation kit), clean/soiled side	fr	Notice de montage dérivateur de mousse BWS 02 (jeu d'adaptation dérivation buées et mousse) côté propre-côté contaminé
ni	Montage-instructie BWS 02 Schuimcompensator (onderdeel wasem- en schuimafvoer) schone-vuile-zijde	da	Monteringsanvisning BWS 02 skumkompensator (ombyggnings- sæt em- og skumafledning) ren- uren-side	sv	Monteringsanvisning BWS 02 skumkompensator (byggsats för ång- och skumavledning) ren resp. oren sida
it	Istruzioni di montaggio BWS 02 compensatore schiuma (set di montaggio scarico fumane e schiuma) lato asettico-infetto	es	Instrucciones de montaje del compensador de espuma BWS 02 (Juego de montaje de la conducción de vahos y espuma) lado contaminado/ descontaminado.	el	Οδηγία τοποθέτησης BWS 02 ποτενσιόμετρου αφρού (σετ τοποθέτησης για την αποχέτευση υδρατμών και αφρού) καθαρή- ακάθαρτη-πλευρά
am	Installation Instructions - BWS 02 Foam Compensator (Vapor and Foam Dissipation Kit), Clean/Soiled Side				

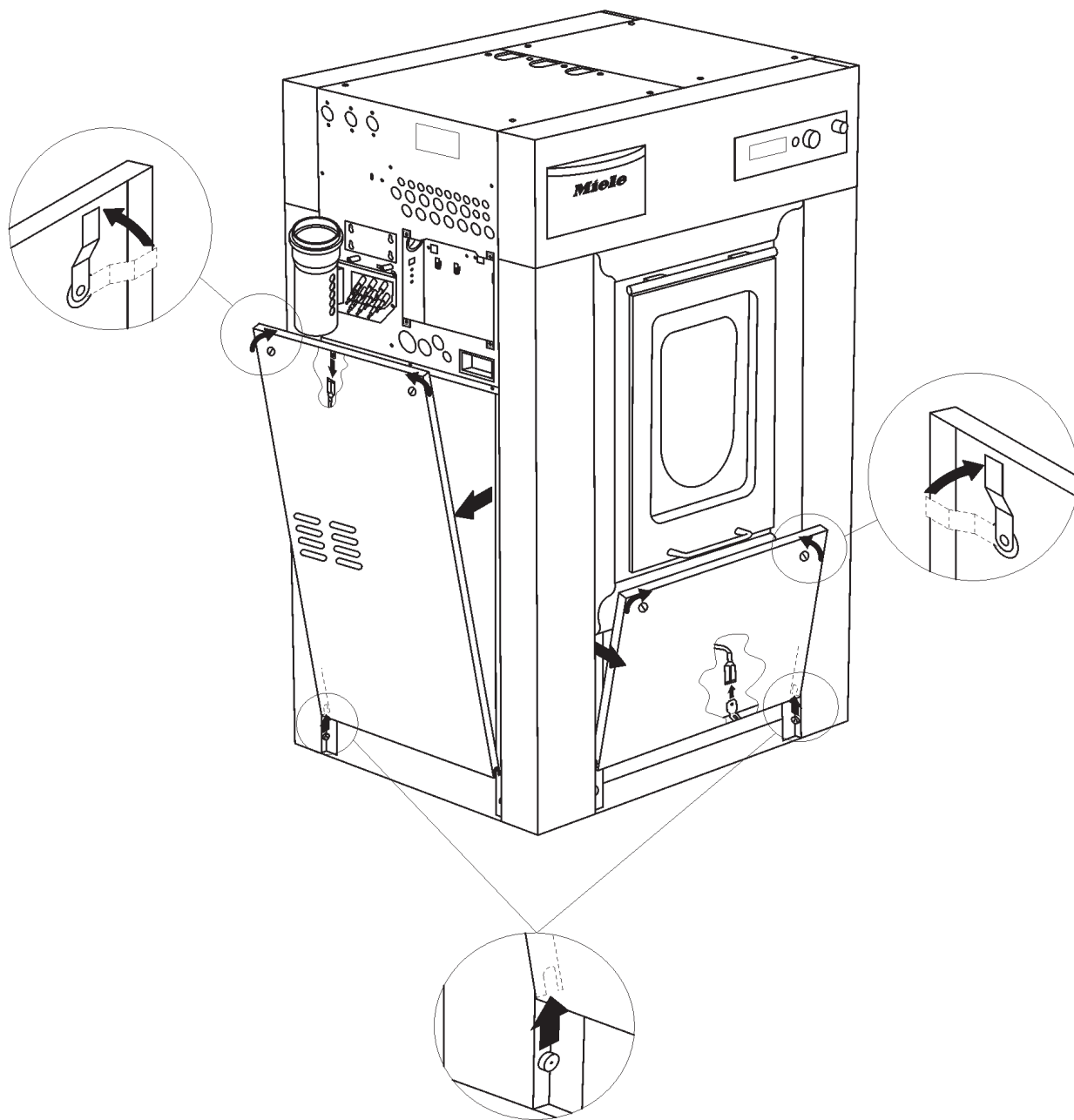
1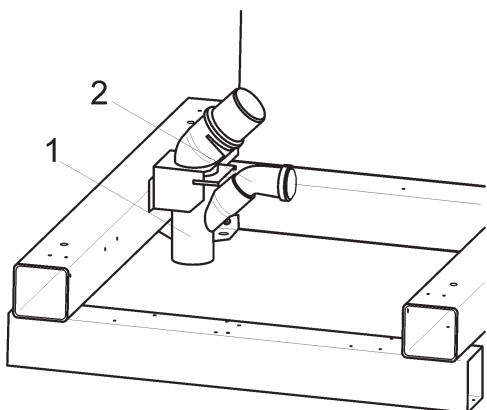


2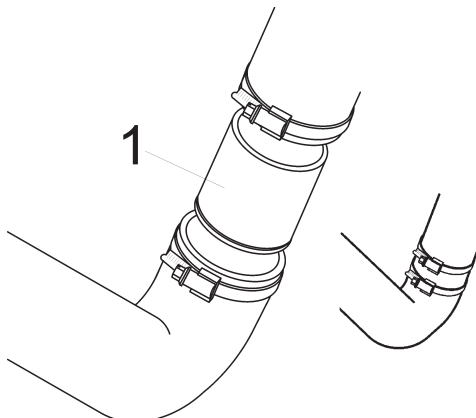


3

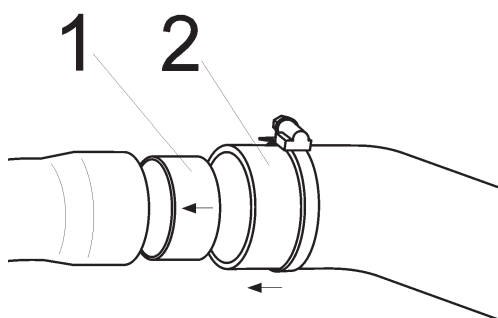
4

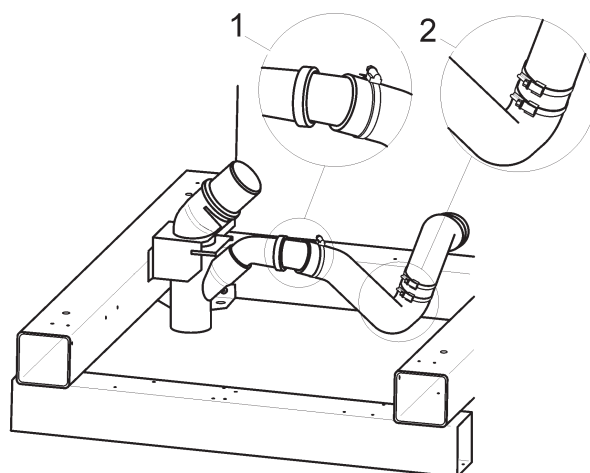
5

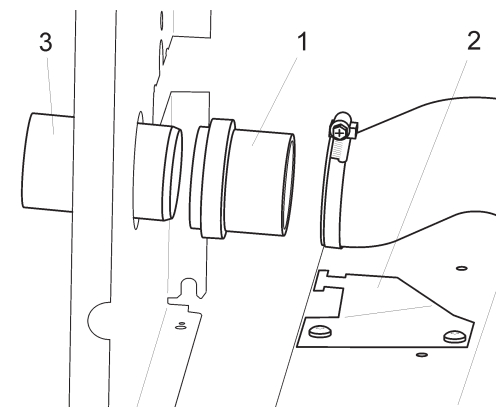
6

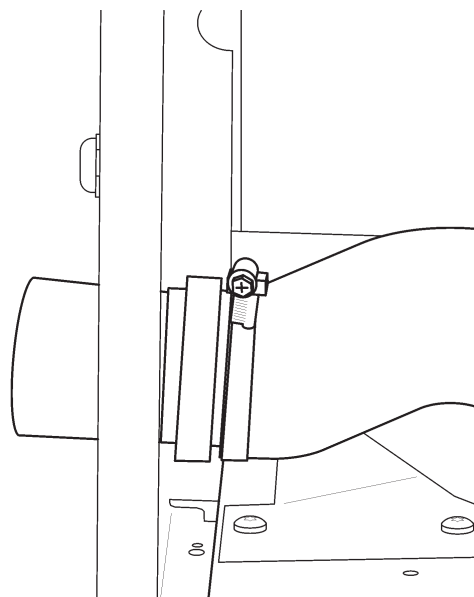
7

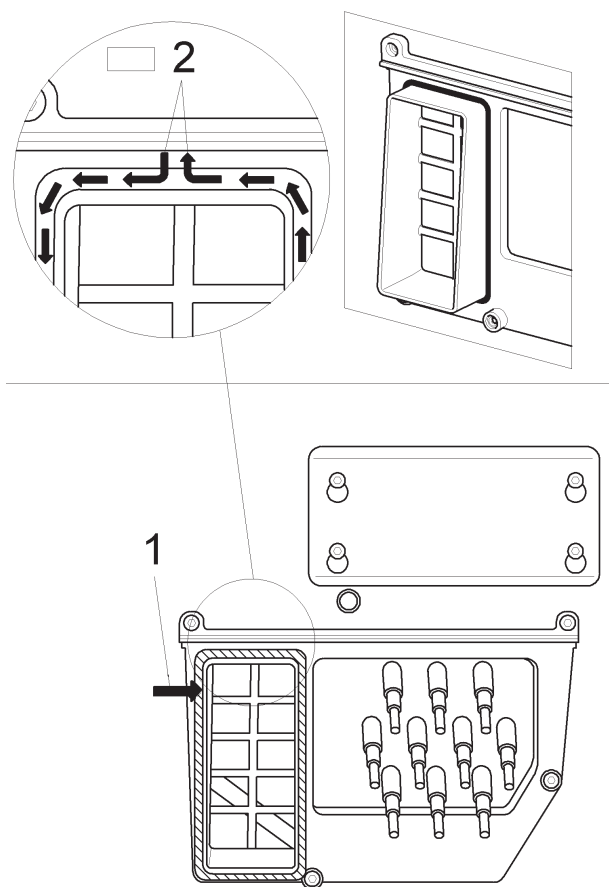8

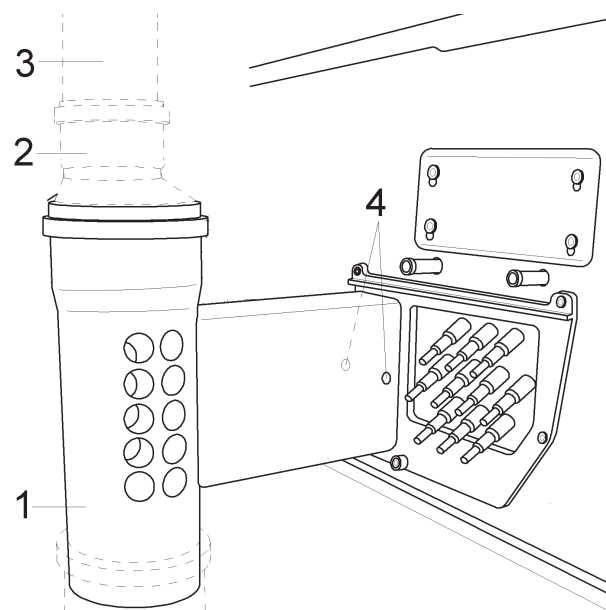9

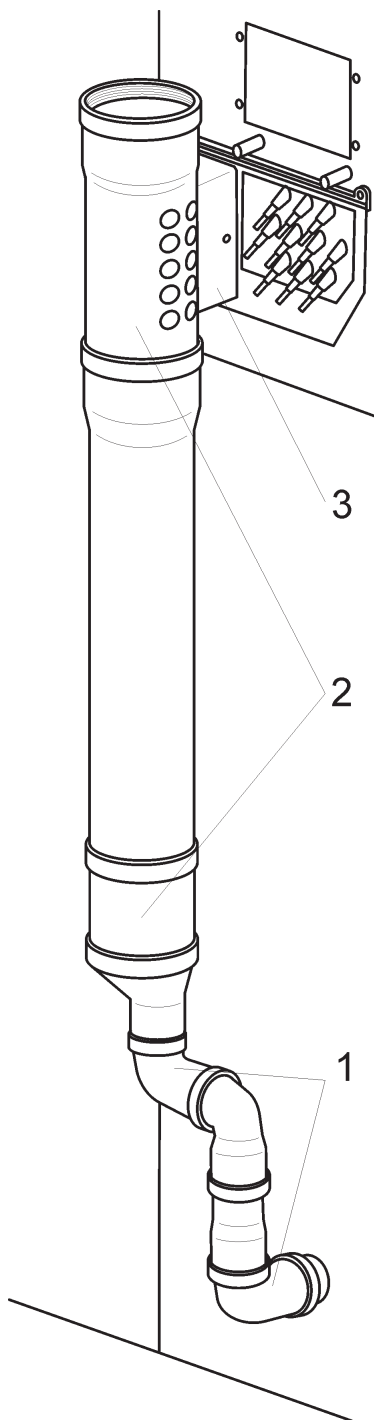10

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Danger!

Even with the machine switched off, mains voltage may be applied to some components.

Before any service work is commenced, the machine must be disconnected from the mains. Suitable measurements must be made to ensure that this is the case.

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Note

The kit, Mat. no. 07555850, for PW 6163, PW 6243 and PW 6323, contains the following:

- 1 branch pipe 75 x 50, 45°
- 1 pipe bend 50 x 45°
- 4 hose clips SGL 50-70
- 1 hose I
- 1 hose II
- 1 connection piece
- 1 pipe bend DN 50 x 45° (pipe bend hose fastening)
- 1 ring (reinforcing ring for pipe bend hose fastening)
- 1 connection piece 1
- 1 connection piece 2
- 1 holding plate
- 2 raised-head screws CE M6 x 16
- 1 adapter for vapour/foam dissipation
- 2 raised-head screws 4 x 15
- 1 foam adhesive tape 0.5 m
- 1 pipe 110 x 500
- 1 pipe 110 x 50
- 1 sleeve 110
- 3 bends 50 x 87
- 1 pipe 50 x 100
- Fitting instructions - BWS 02 foam compensator (vapour and foam dissipation kit), Mat. no. 07553950.

List of illustrations:

- Fig. 1: Side casing removal
- Fig. 2: Branch piece fitting, view from right
- Fig. 3: Hose I and hose II pre assembly
- Fig. 4: Pipe bend hose fixing pre assembly
- Fig. 5: Drain connection, view from right
- Fig. 6: Side casing through-feed fitting
- Fig. 7: Side casing fitting
- Fig. 8: Fitting foam adhesive tape

- Fig. 9: Fitting, drilling and fastening the vapour and foam dissipation adapter
- Fig. 10: Pipe combination

- Disconnect the machine from the electrical mains and ensure power cannot be switched on again in error.
- Remove the soiled side casing and the left side casing panels.
- Connect the branch pipe 75 x 50, 45°, Fig. 2, Pos. 1, with the pipe bend 50 x 45°, Fig. 2, Pos. 2, and slide them into position on the fixing bracket on the machine drain pipe.
- Fit hose I and hose II on the connection piece, Fig. 3, Pos. 1, slide the parts fully together and fit the hose clips SGL 50-70. For the final fitting position, see Fig. 5.

Note

To simplify drain removal/fitting, ensure that the hose clips SGL 50-70 face towards the left side casing.

- Fit the reinforcing ring, Fig. 4, Pos. 1, in the pipe bend hose fixing.
- Fit the pre assembled hose I and II assembly, Fig. 4, Pos. 2, on the pipe bend hose fixing and fit a hose clip SGL 50-70. For the final fitting position, see Fig. 5.

Note

To simplify drain removal/fitting, ensure that the hose clips SGL 50-70 face towards the left side casing.

- Slide the pipe bend hose fastening into the pipe bend 50 x 45°, Fig. 5, Pos. 1.
- Fit connection piece 1, Fig. 6, Pos. 1, in hose I.
- Attach the holding plate, Fig. 6, Pos. 2, to the chassis with 2 raised-head screws CE M6 x 16. Use the existing holes in the chassis.
- Screw connection piece 1 and hose I with a hose clip SGL 50-70 to the holding plate.
- Fit connection piece 2, Fig. 6, Pos. 3, into connection piece 1.
- Break out the pre-cut piece in the side casing.
- Deburr the hole.
- Fit the left side casing, Fig. 7.
- Carefully clean and dry the mixer box around the vent, Fig. 8, Pos. 1, (hatched area) before fitting the foam adhesive tape to ensure that it is attached securely.
- When sticking the adhesive tape in place, start at the top in the middle, Fig. 8, Pos. 2. Then apply the tape in the direction of the arrow around the vent collar.
- Depending on the on-site conditions, prepare the existing vent system. For example cut the pipe pieces to size and fit them together, Fig. 9, Pos. 2, Pos. 3.
- Fit the adapter, Fig. 9, Pos. 1, on the collar of the suds container vent.
- Through the holes in the adapter, drill 2.5 mm dia. holes on the left and right in the suds container vent collar, Fig. 9, Pos. 4.
- Pre assemble the pipe bend combination, Fig. 10, Pos. 1, consisting of 3 x bend 50 x 87 and 1 x bend 50 x 100 and fit it on connection 2 on the side panel.
- Pre assemble the pipe combination, Fig. 10, Pos. 2, consisting of 1 x sleeve 110, 1 x pipe 110 x 50, 1 x pipe 110 x 500 and adapter for vapour/foam dissipation.
- Fit the pre assembled pipe combination, Fig. 10, Pos. 2, on the pipe bend combination, Fig. 10, Pos. 1, and the suds container vent collar, Fig. 10, Pos. 3.
- Fasten the adapter to the vent collar with the two 4 x 15 mm raised-head screws provided, Fig. 9, Pos. 4.
- Connect the adapter to the on-site vent system. If the vent connection, Fig. 9, Pos. 2, is not being used, the appropriate opening must be closed with a stopper. The small openings (10 each on the right and left) in the adapter must remain open.
- Carry out a test run and check the vapour/foam dissipation system for leaks.
- Refit the side casing on the soiled side.
- Carry out appropriate electrical safety checks.
- Reconnect the machine to the electric mains supply.

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Note

This **service and repair work** should only be carried out by a suitably qualified electrician (with specialist training, knowledge and experience, and recent related work experience) in accordance with all appropriate local and national safety regulations.

Servicing, modification, testing and maintenance of electrical appliances should only be carried out in accordance with all appropriate legal requirements, accident prevention regulations and valid standards. All regulations of the appropriate utility supply companies and standards relating to safety (not limited to electrical safety) are to be complied with.

Danger!

Even with the machine switched off, voltage may be applied to some components.

Before starting any service work, the machine must be disconnected from the power supply. Suitable measurements must be made to ensure that this is the case.

A general visual check should always be carried out.

Incorrect conversion or service work could lead to a risk of fire.

Note

This kit, mat. no. 07555850, for PW 6163, PW 6243 and PW 6323, contains the following:

- 1 branch pipe 75 x 50, 45°
- 1 pipe bend 50 x 45°
- 4 hose clips SGL 50-70
- 1 hose I
- 1 hose II
- 1 connection piece
- 1 pipe bend DN 50 x 45° (pipe bend hose fastening)
- 1 ring (reinforcing ring for pipe bend hose fastening)
- 1 connection piece 1
- 1 connection piece 2
- 1 retaining bracket
- 2 raised-head screws CE M6 x 16
- 1 adapter for vapor/foam dissipation
- 2 raised-head screws 4x15
- 1 foam adhesive tape 0.5 m
- 1 pipe 110x500
- 1 pipe 110x50
- 1 sleeve 110
- 3 bends 50x87
- 1 pipe 50x100
- These installation instructions

List of illustrations:

- Fig. 1: Side casing removal
- Fig. 2: Branch piece installation, view from right
- Fig. 3: Hose I and hose II pre-assembly
- Fig. 4: Pipe bend hose mounting pre-assembly
- Fig. 5: Drain connection, view from right
- Fig. 6: Side casing feed-through installation
- Fig. 7: Side casing installation
- Fig. 8: Installing foam adhesive tape

- Fig. 9: Installing, drilling and fastening the vapor and foam dissipation adapter
- Fig. 10: Pipe combination

- Disconnect the machine from the electrical supply and ensure that power cannot be switched on again in error.
- Remove the soiled-side casing and the left side casing panels.
- Connect the branch pipe 75 x 50, 45° (Fig. 2, Pos. 1) to the pipe bend 50 x 45° (Fig. 2, Pos. 2) and slide them into position on the mounting bracket on the machine drain pipe.
- Install hose I and hose II on the connection piece (Fig. 3, Pos. 1), slide the parts fully together and install the SGL 50-70 hose clips. For the final installation position, see Fig. 5.

Note

To simplify drain removal/installation, ensure that the hose clips face towards the left side casing.

- Install the reinforcing ring (Fig. 4, Pos. 1) in the pipe bend hose assembly.
- Install the hose I and II assembly (Fig. 4, Pos. 2) on the pipe bend hose assembly and install a hose clip. For the final installation position, see Fig. 5

Note

To simplify drain removal/installation, ensure that the hose clips face towards the left side casing.

- Slide the pipe bend hose fastening into the pipe bend 50 x 45°, Fig. 5, Pos. 1.
- Install connection piece 1 (Fig. 6, Pos. 1) in hose I.
- Attach the mounting bracket (Fig. 6, Pos. 2) to the chassis with 2 raised-head screws. Use the existing holes in the chassis.
- Screw connection piece 1 and hose I to the mounting bracket with a hose clip.
- Install connection piece 2 (Fig. 6, Pos. 3) in connection piece 1.
- Break out the pre-cut piece in the side casing.
- Deburr the hole.
- Install the left side casing, Fig. 7
- Carefully clean and dry the mixer box around the vent (Fig. 8, Pos. 1, hatched area) before installing the foam adhesive tape to ensure that it is attached securely.
- When sticking the adhesive tape in place, start at the top in the middle, Fig. 8, Pos. 2. Then apply the tape in the direction of the arrow around the vent collar.
- Depending on the on-site conditions, prepare the existing vent system. For example, cut the pipe pieces to size and fit them together, Fig. 9, Pos. 2, Pos. 3.
- Install the adapter (Fig. 9, Pos. 1) on the collar of the suds container vent.
- Through the holes in the adapter, drill 2.5 mm dia. holes on the left and right in the suds container vent collar, Fig. 9, Pos. 4.
- Pre-assemble the pipe bend combination (Fig. 10, Pos. 1), consisting of three 50 x 87 bends and one 50 x 100 bend and install it on connection 2 on the side panel.
- Pre-assemble the pipe combination (Fig. 10, Pos. 2), consisting of one 110 sleeve, one 110 x 50 pipe, one 110 x 500 pipe and adapter for vapor/foam dissipation.
- Install the pre-assembled pipe combination (Fig. 10, Pos. 2) on the pipe bend combination (Fig. 10, Pos. 1) and the suds container vent collar (Fig. 10, Pos. 3).
- Fasten the adapter to the vent collar with the two 4 x 15 mm raised-head screws provided, Fig. 9, Pos. 4.
- Connect the adapter to the on-site vent system. If the vent connection (Fig. 9, Pos. 2) is not being used, the appropriate opening must be closed with a stopper. The small openings (10 each on the right and left) in the adapter must remain open.
- Carry out a test run and check the vapor/foam dissipation system for leaks.
- Reinstall the side casing on the soiled side.
- Carry out appropriate electrical safety checks.
- Reconnect the machine to the electrical supply.

am