Areas of application

Separable MIS instruments with a shaft up to 420 mm in length can be processed in insert E 908/1.

Insert E 908/1 can be hung in modular inserts for MIS instruments, e.g. an E 906/1. The modular inserts come with their own operating instructions.

 Please observe the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. You should only process instruments/medical devices in this insert if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the medical devices to be processed are made. Where possible the programme should include thermal disinfection.


MIS instruments should be taken apart for processing where possible. The shafts of the instruments are placed in injector sleeves, and the handles fitted to the correct jets in the modular insert.

Arranging the separable MIS instruments

Separable MIS instruments should be secured at three or four points in the notches in the E 908/1. The notched bars can be adjusted to the length of the instruments for this purpose.

- Arrange the instruments in the notches.
- First of all secure the functioning end of the instrument, and make sure that it is open.
- Then secure the instrument in the other notches.

Optics with a diameter of up to 5 mm, e.g. for urology, can also be processed in this insert. To ensure they do not move during processing, they should be secured in four notches.

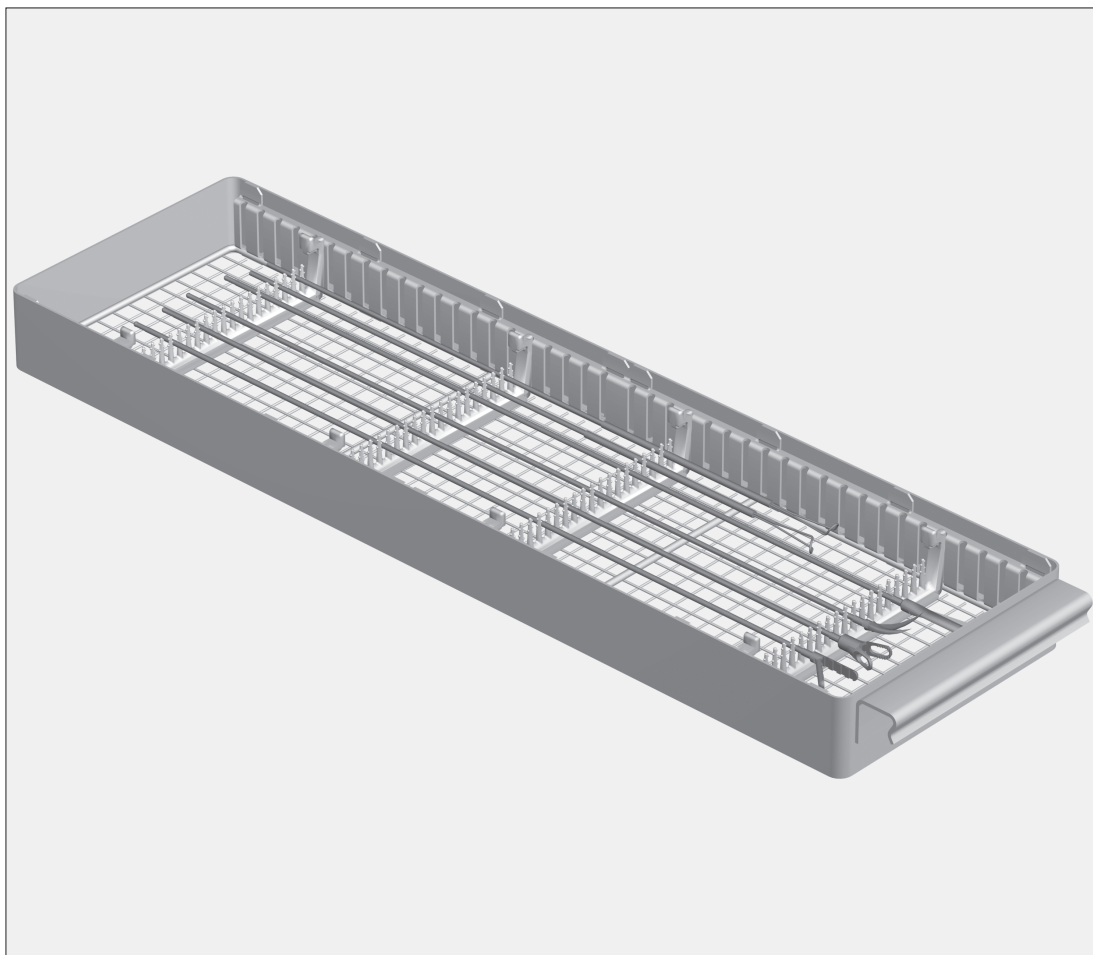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