Areas of application:

The E 427 insert can be used for cleaning and disinfecting up to six laryngoscopes in most sizes.


The E 427 insert is suitable for use in injector mobile units for anaesthetic instruments, e.g.:

- E 435, E 436, E 461,
- E 515, E 715, E 915.

⚠ Please read the Warning and Safety instructions in the Operating Instructions for your washer-disinfector carefully. You should only process laryngoscopes/medical products in this insert if they have been declared as suitable for machine processing by the manufacturer. Please also observe the manufacturer's care instructions. The programme selected must be suitable for the materials from which the medical products to be processed are made. Where possible the programme should include thermal disinfection.

- Hook the laryngoscope onto the upper rail in the insert by the ball joint. The other end of the laryngoscope should, depending on size, be supported on one of the other rails.
- Laryngoscopes should always be arranged in between two rails so that they do not come into contact with each other.
- The E 427 insert should be placed in the flat area at the back of the AN mobile unit. The spatula end of the laryngoscopes should point towards the front of the wash cabinet or into the centre of the cabinet from one side. Only by doing so can critical areas be properly accessed by the wash action.

⚠ To avoid material damage to laryngoscopes they should only be processed in the Vario TD programme with the final rinse and desinfection being carried out with fully de-mineralised water.

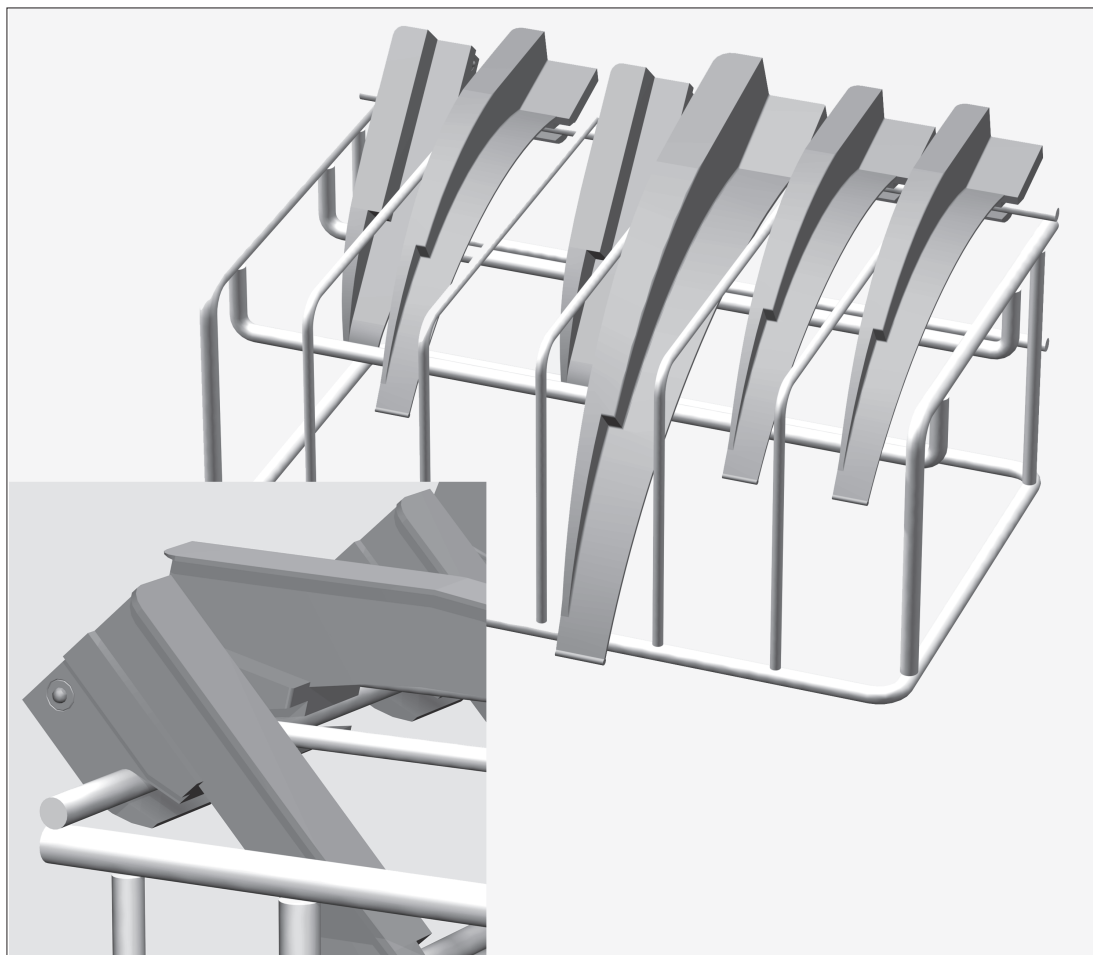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